



IMMULEXTM STREP-B

GBS type antisera Result in 5-10 seconds Easy typing of GBS SSI Diagnostica A/S

Herredsvejen 2 3400 Hillerød Denmark T +45 4829 9100

ssidiagnostica.com

BACKGROUND

The Immulex $^{\text{M}}$ Strep-B kit is simple and fast to use for serotyping of *S. agalactiae* (GBS). Using the kit, the commonly isolated GBS isolates can be serotyped and placed within all 10 known serotypes (Ia, Ib, - IX).

APPLICATION

Overnight bacterial cultures of GBS isolates in Todd-Hewitt broth are mixed with the Immulex $^{\text{\tiny{M}}}$ Strep-B suspension.

DESCRIPTION

The 10 latex suspensions are ready to use and delivered in 1.5 ml plastic vials with a droplet delivery system, administering approximately 10 μ l. The antibodies in the latex suspension are raised in rabbits against the capsular polysaccharide present in each solution. The latex suspension is preserved with 15 mM (0.0975%) sodium azide.

PROCEDURE

The latex agglutination test is performed as follows:

- 1 droplet (10 µl) of bacterial suspension is placed on the plastic slide.
- 1 droplet (10 μl) of latex suspension is also placed on the plastic slide.
- The two suspensions are mixed.

A positive reaction is indicated by an agglutination appearing within 5 to 10 seconds.

If the reaction time exceeds 30 seconds, false positive reactions may occur.

LIMITATIONS

The Immulex[™] Strep-B kit is intended for typing of pure cultures of GBS isolates, preferably grown in Todd-Hewitt broth.

STORAGE AND SHELF LIFE

The kit should be kept in a dark place at 2-8°C. The expiry date is printed on the package.

INFORMATION AND ORDERING

SSI Diagnostica A/S Herredsvejen 2 3400 Hillerød T 4829 9100 info@ssidiagnostica.com ssidiagnostica.com shop.ssidiagnostica.com